

DEVICE DESCRIPTION

Roger Pen is a microphone that picks up the voice of a speaker and sends it wirelessly directly into your hearing devices. You can place Roger Pen in the middle of a table e.g. in a restaurant, or it can be worn by a single person i.e. to hear a distant talker. Follow the steps below to pair the Roger Pen.

PAIRING ROGER PEN TO A PROCESSOR OR HEARING AID

- 1 Turn on the Roger Pen. Ensure the appropriate battery to the Roger receiver is attached to the processor or hearing aid.
- 2 Hold Roger Pen within 10 cm/4 inches from the Roger receiver or hearing aid.
- 3 Press the “Connect” button  on the side of the Roger Pen.

✓ TIP

The front indicator light will turn green for two seconds to confirm a successful connection.

You can connect an unlimited number of Roger receivers to your Roger Pen, simply repeat step 2 and 3 for any other Roger receiver.

	Blue Blinking	Connecting or pairing mode
	Solid Green	Connecting/pairing successful
	Red Double Blinking	Connect timeout, move device closer
	Red Triple Long Blinking	Device not compatible


STANDBY MODE

- 1 To enter standby mode, press and hold the on/off button to put Roger Pen into Bluetooth standby mode. This means the Roger Pen microphone is switched off but incoming calls can still be received.
- 2 In Bluetooth standby mode, the battery of the Roger Pen is drained. It is recommended to switch off the Roger Pen completely if no calls are expected to be made.

  **Switching off, Bluetooth in standby**

PAIR ROGER PEN WITH YOUR PHONE/ COMPUTER (BLUETOOTH)

- 1 Turn on the Roger Pen and phone/computer. Place next to each other.
- 2 Ensure the phone/computer Bluetooth setting is enabled.
- 3 On the phone/computer, choose SEARCH or SCAN to search for Bluetooth enabled devices or audio enhancements.
- 4 Press the Accept button for 7 seconds until the indicator light blinks blue.

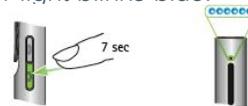

- 5 The phone/computer should present a list of Bluetooth devices it has detected. Choose “Roger Pen” from this list and confirm the connection is established.
- 6 The Roger Pen should now be paired to the phone/computer and the indicator light will stop blinking.

✓ TIP

The phone/computer may ask to enter a passcode. If so, enter '0000'. Some phones/computers may ask which Bluetooth service you want to enable, select Headset audio device and microphones in parallel.

	Blue Slow Blinking	Bluetooth Standby Mode
	Blue Solid	Incoming Call
	Blue Solid	Active Call
	Blue Fast Blinking	Network Issue, please press the connect button

PHONE CALLS

Accepting Incoming Calls - The Roger Pen Bluetooth light indicator will be solid blue when receiving a call. To accept an incoming call, press the Accept button.

Making Phone Calls - Once you dial on your phone, you will hear the ring tone in your processor/hearing aid. It is recommended to wear your Roger device on your chest during the phone call or place it on the table in front of you.

Ending a Call – Press the Decline button and the call will terminate.

Rejecting a Call – Press the Decline button. You can also reject the call from your phone.

Mute Call - To mute your voice, firmly press the on/off button. The purple mute indicator light will light up. **Press the on/off button again to unmute your voice**

LISTENING TO A PORTABLE AUDIO DEVICE

- 1 Make sure you have switched on the Roger Pen
- 2 Plug the rectangular end of the short audio cable into the Roger Pen.
- 3 Plug the round end of the short audio cable into the headphone output of your portable device.
- 4 Start the music or music app on the portable audio device.

✓ TIP

The Roger Pen microphone will not automatically mute when the audio cable is plugged in. To mute the microphone, press the on/off button and the purple indicator light will appear.

To end the transmission of the audio signal, unplug the audio cable or switch the Roger Pen off.



BATTERY STATUS

	Solid Orange	Charging
	Green Solid	Fully Charged
	Green Slow Blinking	Switched on, Battery Full
	Dimming Red	Powering Off
	Red Blinking	Battery Empty, Charge Roger Pen

MICROPHONE MODES

Roger Pen detects which situation you are in and adapts its microphone setting automatically to provide you the best speech understanding possible.


Conference Style - Place Roger Pen in the middle of a table to listen to a group of people.



Interview Style – Hold Roger Pen in your hand and point it towards the person you want to hear.



Around the Neck – Use the lanyard (neckloop) to hang the Roger Pen around the neck of the speaker you want to hear.

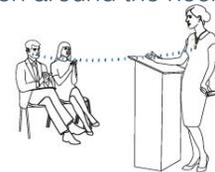

**CAN'T FIND WHAT YOU NEED?
CONTACT US!**



877-829-0026



Contact Us